

PHILIPS

sense **and** simplicity

RADIOMETER 

IntelliVue TcG10 Sales Training
November 2011

Rebecca Hill and Phil Lazzara

IntelliVue TcG10




IntelliVue TcG10 – Key Benefits

- State of the art TC-Gas solution with integrated and automated calibration supporting clinical workflow
- Improved, enhanced algorithms for use in Neonate, Pediatric, Adult patients
- Smaller sensor, lower temperatures, easier to apply
- Configuration choices in IntelliVue Monitor
- Integration into the IntelliVue patient monitor via IntelliBridge
- Self contained cal gas
- Standardized mounting solutions



New IntelliVue TcG10 Solution - Description

- TcGas measures continuous transcutaneous partial pressure of O₂ and CO₂ in neonate, pediatric, and adult
- Compliments a discreet Arterial Blood Gas measurement
- The IntelliVue TcG10 is connected to the host monitor via the IntelliBridge EC10 module.
- The IntelliVue TcG10 is compatible with IntelliVue monitors MP40-90, MX700/800 with software Rev. H or higher, and use IntelliBridge modules

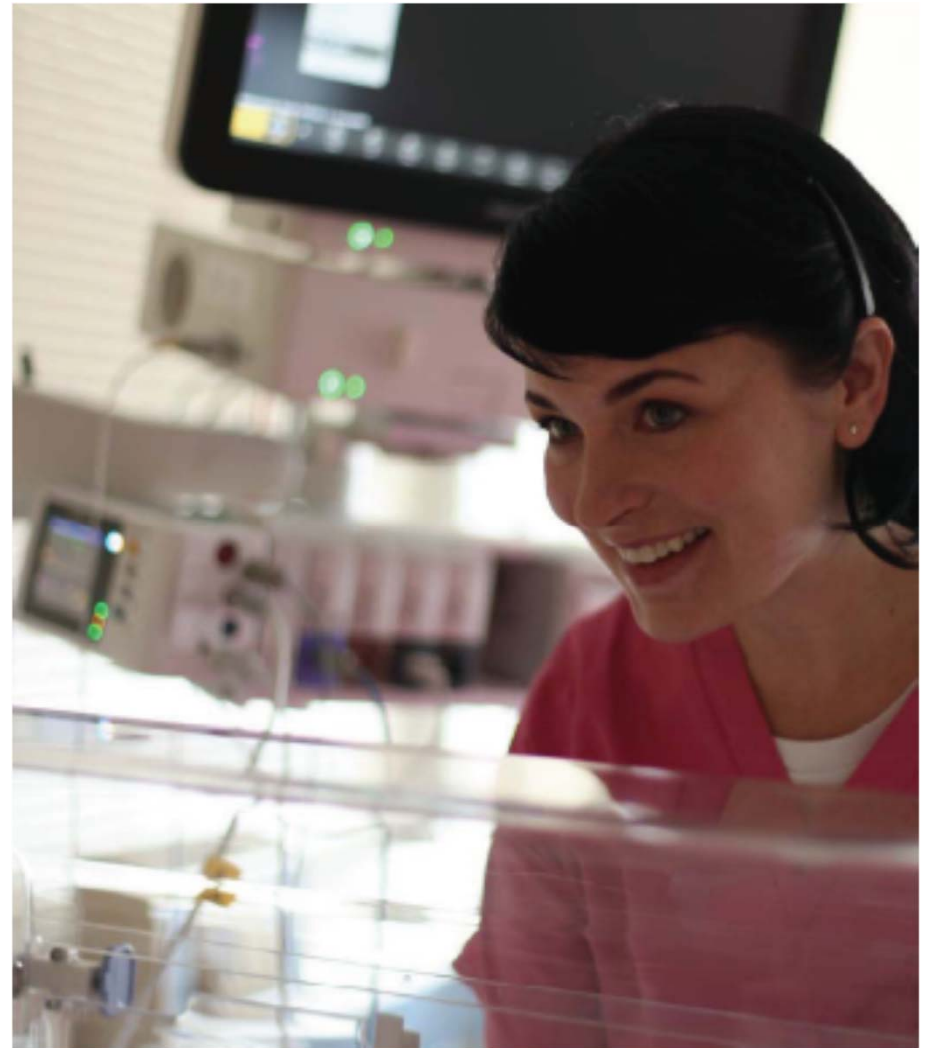


IntelliVue TcG10: Key Features




- Ready to use
 - no set-up time
- Automatic calibration
 - no manual handling
- Stable/robust sensor
 - membrane protection
- Easy and fast re-membraning
 - using Fit & Click
- 14 days re-membraning interval
 - minimal handling
- Reliable measurement
 - automatic check of sensor performance
- Sensor memory for performance data and date of re-membraning
 - indication of re-membraning request

TcG10 – User Interface in the IntelliVue Monitor

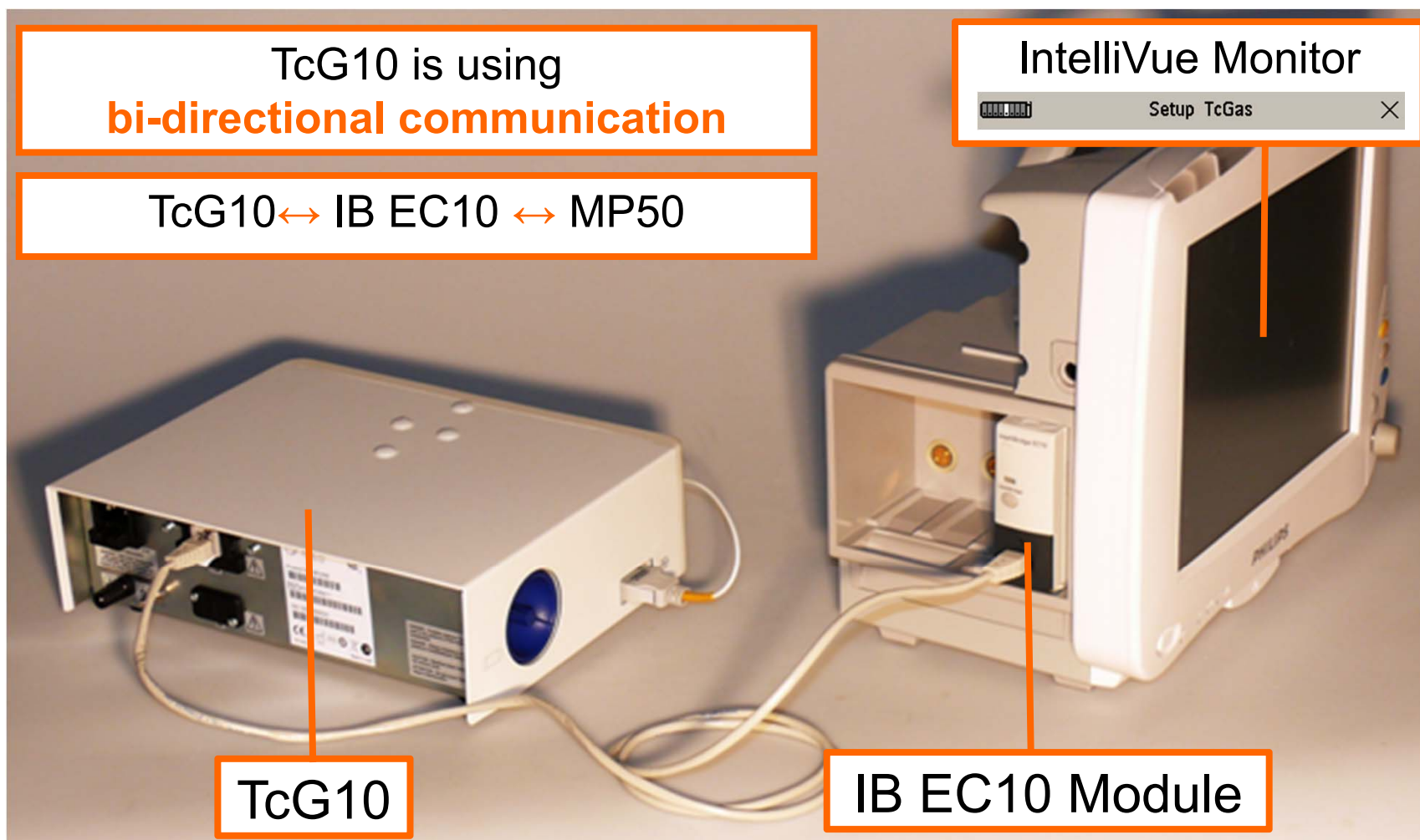

TcG10 Data on the Screen

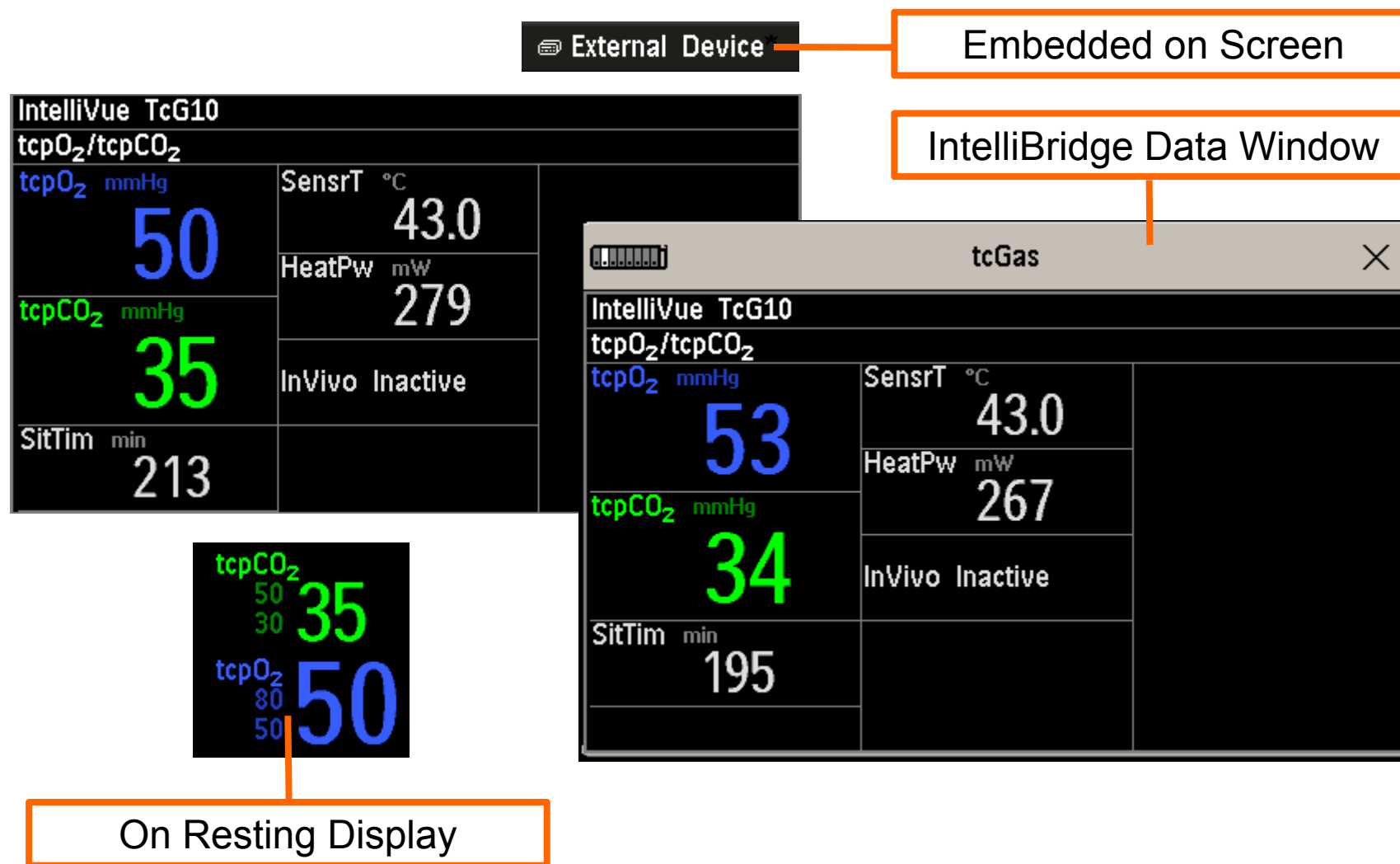

Available Parameters

Label	Measurement Description	Unit
tcpO2	Transcutaneous partial pressure of oxygen (tcpO ₂)	mmHg, kPa
tcpCO2	Transcutaneous partial pressure of carbon dioxide (tcpCO ₂)	mmHg, kPa
SitTim	Site time	min
SensrT	tc Sensor 84 temperature	°C, °F
HeatPw	tc Sensor 84 heating power (Absolute, Relative)	mW
InVivo	In vivo correction mode	Inactive, Active

Measured Values

Information Message

Quote & Ordering

- IntelliVue TcG10 Product Number: **865298**
- Options

<ul style="list-style-type: none"> - K05 (standard) - K10 (optional) 	<p>EC 5 with LAN Cable 1.5 m</p> <p>EC10 IntelliBridge Module driver loaded for IntelliVue</p>
--	--
- Quickmount Solution (Tabletop Mount) is Standard
- Availability



FDA and Health Canada: US November 2011 price book

M1018A TcGas Solution



- Uses Radiometer sensor M1918A/B
 - Sensor and module reaching its end of life
- Requires external calibration unit (15210B)
- Requires separate cal gas
- Early 1990's technology (15+ years old)
 - Solution is difficult to use and perceived as being complicated
 - Manual step for user
 - Cumbersome, lengthy calibration and stabilization
 - Support constraints - parts becoming obsolete and difficult to get
- Marketing Minute 439 announces product discontinuance plan for M1018A
 - Note: existing sensors, M1918A and M1918B are not part of this discontinuance

Collaborative effort for Philips and Radiometer Contracted Partnership


PHILIPS

sense **and** simplicity

RADIOMETER 

Simpler, faster, better

Partnership - How to Work Together

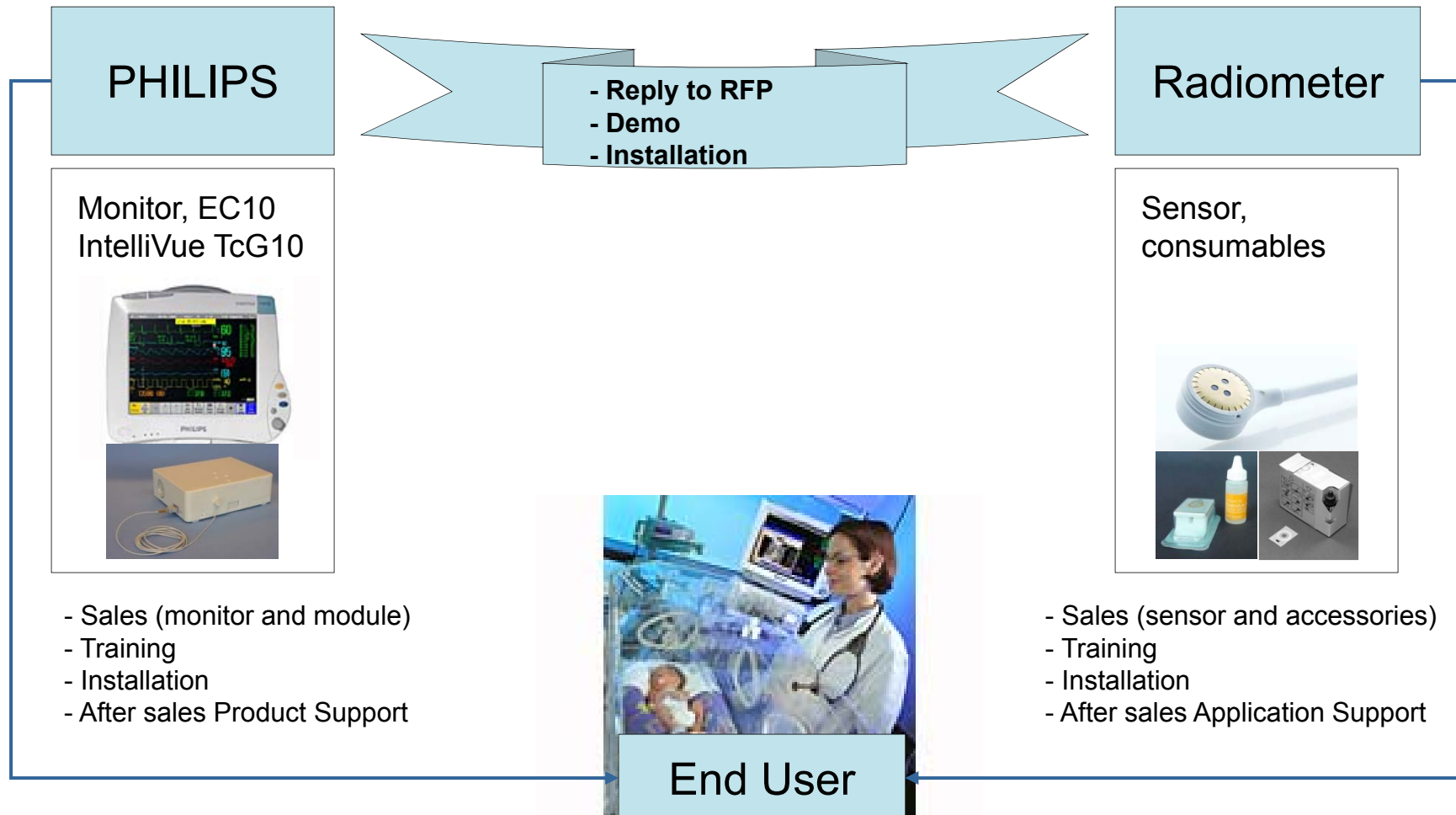

Sales, Training and Support

Overall Strategy

Collaborative between Philips and Radiometer

Sales, Pre-sales Clinical Trials, Post-sales Training, Go-Live support

- Radiometer provides
 - the sensor 84, consumable supplies and accessories
 - Application training and measurement support to customer
- Philips provides
 - the TcG10 module, IntelliBridge products
 - Installation and training of the monitoring solution
 - Customer Care phone support with escalation to Radiometer if needed

TcGas Application & Sensor

(Phil Lazzara)




TC Sensor 84



Part Number 945-737


Membraning the Sensor

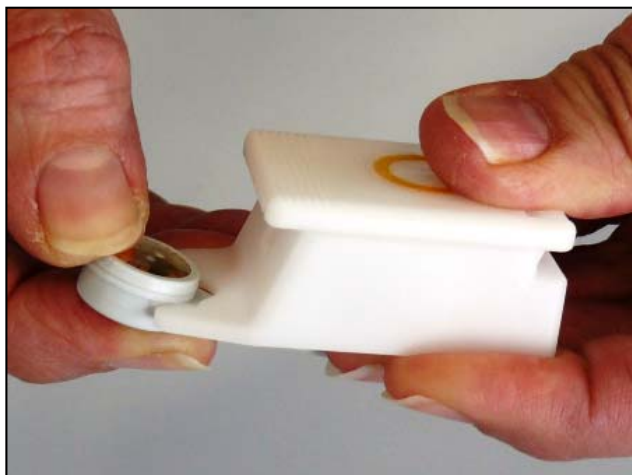

Every two weeks




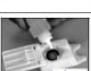


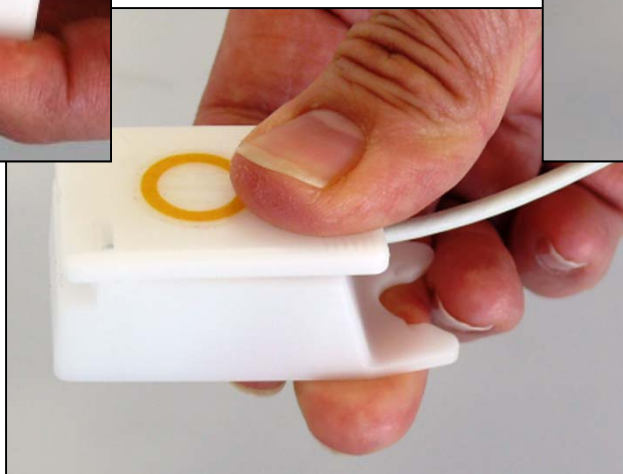



See
Instructions for Use



↑✓ tcGas CHG MEMBRAN


tcpO₂/tcpCO₂ Last prep. 6 Jul 10 15:11, time left: 0h

Membraning Procedure	
1. Ensure that the tc-Sensor B4 is connected to the IntelliVue TeG200, which must be turned on.	
2. Remove the old membrane retainer ring and the spacer (i.e. the membrane that is soaked in the electrolyte), using the U-shaped notch of the preparator base, and discard both.	
3. Clean the tc-Sensor B4 surface with a tissue soaked in demineralized water. This will also remove the spacer (i.e. if it could not be removed in step 2). Dry the sensor surface and make sure that no fibers remain from the tissue. Note: Do not leave a tc-Sensor B4 without membrane in air. Proceed immediately with the next steps. Note: If the INOP tcGas CPBM PREP appears, leave it unattended until you have confirmed the remembraning.	 
4. Lift up the corner of the preparator and place two drops of electrolyte into the corner of the membrane retainer ring.	
5. Insert the tc-Sensor B4 (with its surface pointing downwards) gently into the preparator base until it is tightly locked. Do not apply strong pressure.	
6. Close the corner of the preparator and press it down until the new membrane retainer ring clicks into place.	
7. Open the cover, remove the tc-Sensor B4 and discard the preparator.	

Membraning the Sensor



↑✓ tcGas CFRM PREP


Setup tcGas
×

Site Time	:	4.0 hours
Confirm New Prep		
Decline New Prep		


Please Confirm

To confirm the new preparation, select Confirm

Confirm

Cancel

Membraning the Sensor

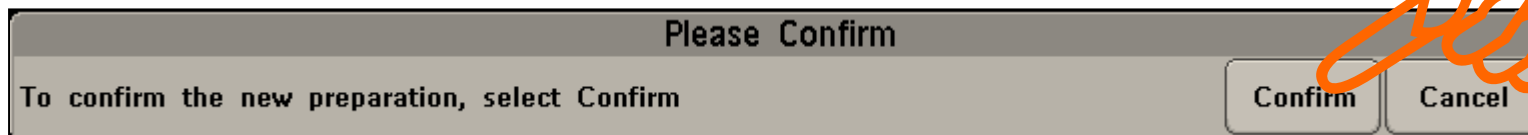

tcpO₂/tcpCO₂ **Sensor preparation done**

tcpO₂/tcpCO₂ Calibration Progress: 73%


tcpO₂/tcpCO₂ **Ready to use**, last cal. 21 Jul 10 10:29

tcpO₂/tcpCO₂ Last prep. 21 Jul 10 10:24, time left: **335h**


Re-membraning of TC Sensor 84

The request for re-membraning is displayed: “tcGas CHG MEMBRAN”


1. Remove ring and membrane

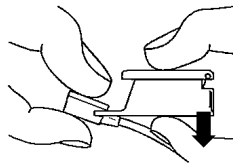


2. Clean with water

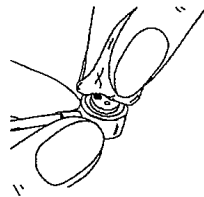


3. Add electrolyte 84

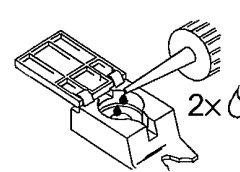


4. Insert sensor face downwards

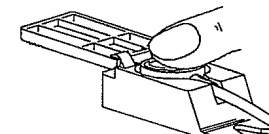


5.

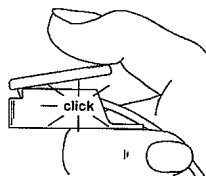


6. Remove excessive electrolyte

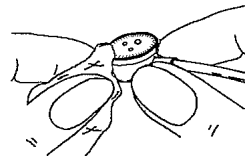

7. Confirm re-membraning “tcGas CFRM PREP”

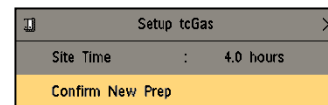

8. Place sensor in cal chamber

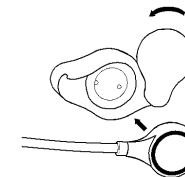


Note: After remembraning a new sensor or a sensor that has not been used for a long time, leave the sensor in the calibration chamber for at least 4 hours to stabilize.

TC Sensor 84 and consumables


tc Sensor 84
part nr. 945-737




Membraning kit 84
(supply for approx. 6 months)
12 preparators, 1x Contact gel
part nr. 905-871

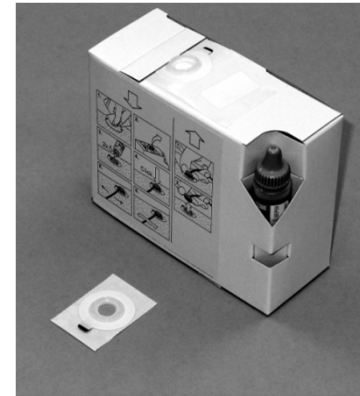

Fixation kit N20
(ø 20 mm for neonates)
60 rings, 1x Contact gel
part nr. 905-873

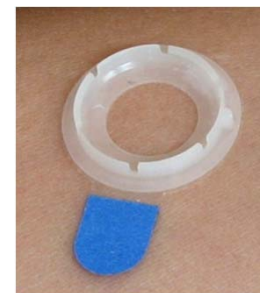


Adhesive rings N20 kit
(ø 20 mm for neonates)
250 rings, 1x Contact gel
Part nr. 905-872

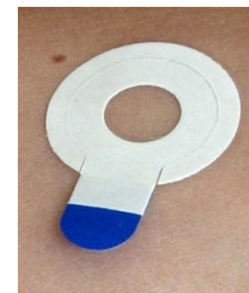


Application / removal of TC Sensor


Fixation ring - application

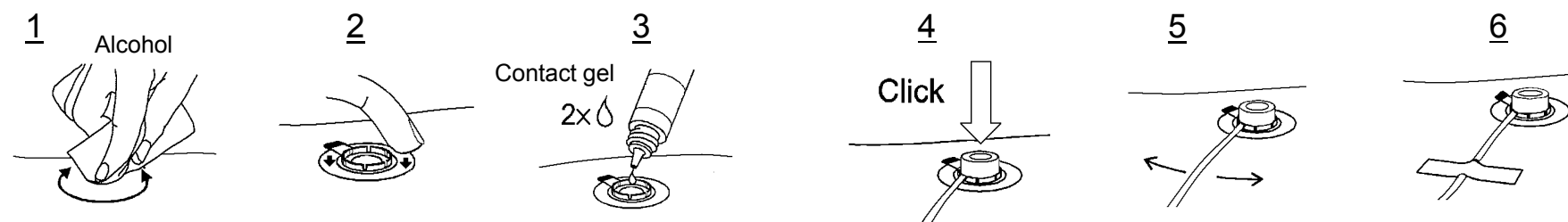



Fixation ring - removal

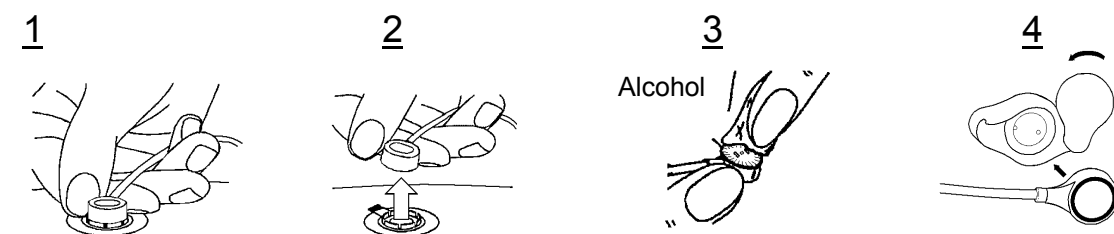



Adhesive ring - application

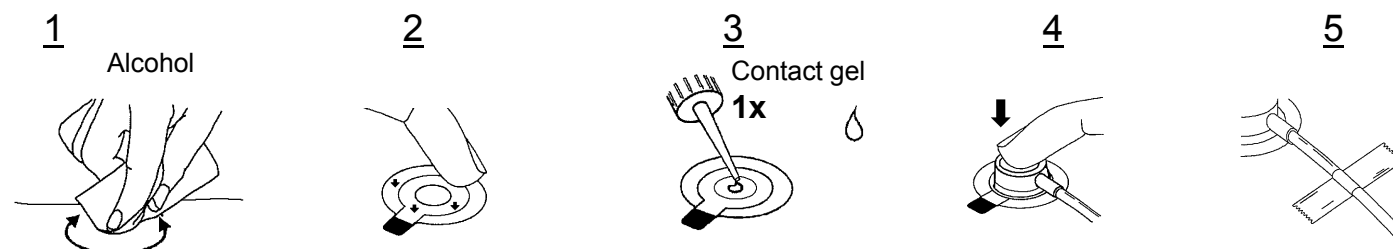

Applying the sensor

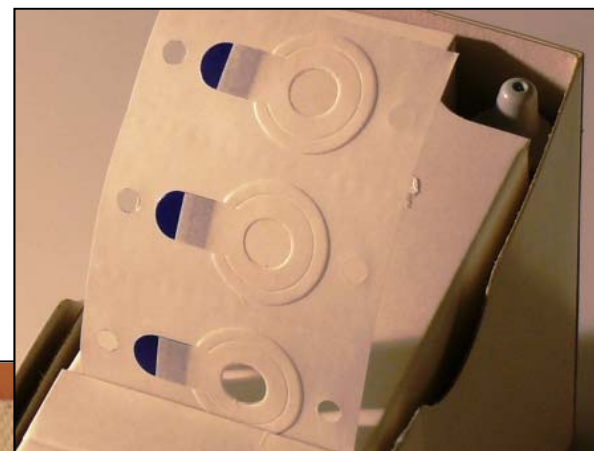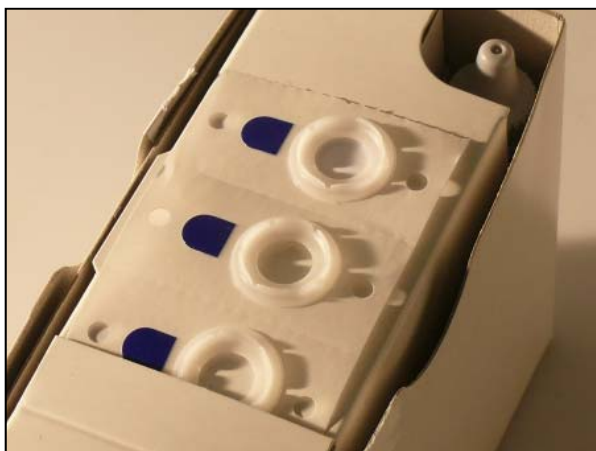

See
Instructions for Use

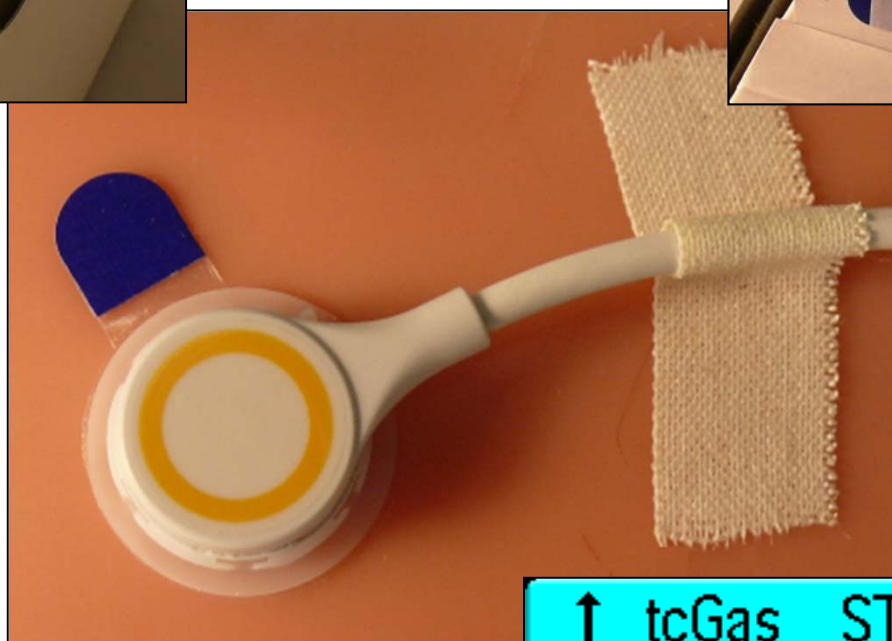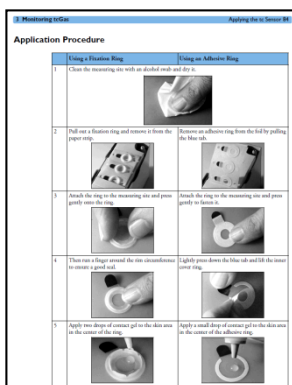


↑ tcGas STABILIZING

*Fluctuations in PaO_2/CO_2 levels
cannot be assessed by
intermittent arterial or capillary
sampling!*

Advanced Technologies Hypocapnia/Hyperoxygenation



SensorMetics 3100B
Adult Oscillator

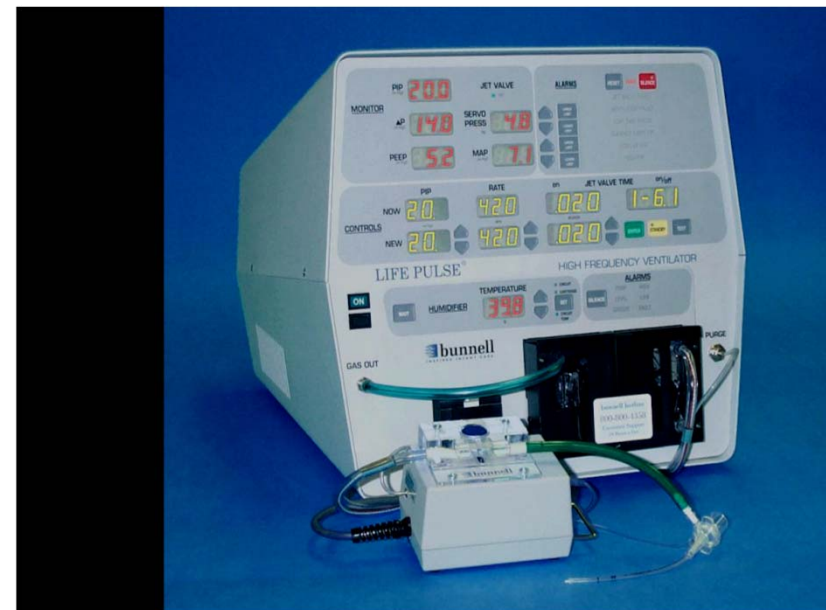

Bunnell Inc. - Neonatal high
frequency jet ventilator

QUESTIONS?